



Topi-CLICK® Patient Use Instructions

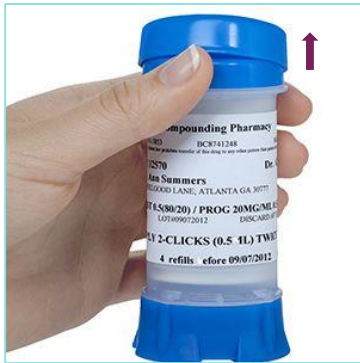

STEP 1:
Pop off the cap of the Topi-Click® container by placing your thumb under the tab on the protective outer cap and pressing upward.




STEP 2:
Turn the base to dispense the number of clicks specified in the instructions on your prescription label.

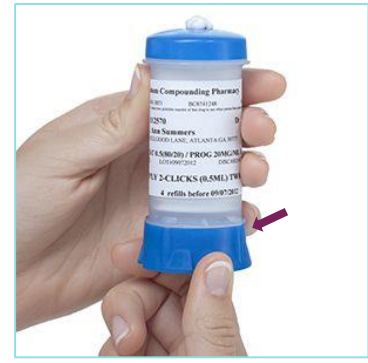

STEP 3:
Watch the visual indicators when rotating base. Be sure to leave the base in the registered click position by aligning the base column with the tube node after each use.



STEP 4:
Apply by rubbing the cream which is on the top of the applicator pad to the area of skin as directed by your pharmacist.

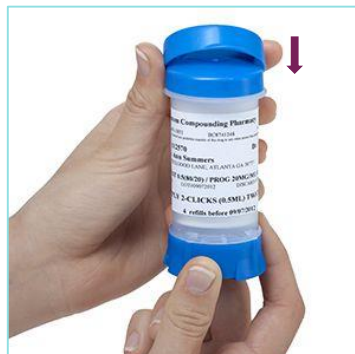

STEP 5:
Replace cap.



STEP 6:
The bottom of the plunger is the indicating line; when it reaches the embossed line below the word "Refill" approximately 32 CLICKs are left. Approximately 12 CLICKs remain at the line above the word "Refill".